



Ref. A22472 DISINFECTANT SPRAY FOR TEXTILES & SURFACES 300ML KING DERM'ALPES

Disinfect textiles and any other surfaces.

- ✓ Delicate fragrance
- ✓ Suitable for disinfection of textiles
- ✓ No irritations for simple skin contact (validated laboratory test)

LIMITED SPECTRUM VIRUCIDAL
Coronavirus, H1N1 (Influenza A)...
And Rotavirus



DISINFECTANT PROPERTIES IN COMPLIANCE WITH STANDARDS AT 20°C, CLEAN CONDITION:

Standards	Strains	Conditions
Bactericidal in compliance with EN 13727	<i>Staphylococcus aureus, Pseudomonas aeruginosa, Escherichia coli, Enterococcus hirae</i>	In 30 seconds
Bactericidal in compliance with EN 13697		In 5 minutes
Fungicidal in compliance with EN 13624	<i>Candida Albicans and Aspergillus brasiliensis</i>	In 30 seconds
Fungicidal in compliance with EN 13697		In 5 minutes
Limited spectrum virucidal in compliance with EN 14476:2015	<i>Murine norovirus, adenovirus and vaccinia virus</i>	In 30 seconds
Additional viruses in compliance with EN 14476:2015	<i>H1N1</i>	In 30 seconds
Additional viruses in compliance with EN 14476 + A2	<i>Rotavirus</i>	In 30 seconds
Mycobactericidal in compliance with EN 14348	<i>Mycobacterium avium complex and mycobacterium terrae (= tuberculocidal activity)</i>	In 5 minutes

APPLICATION

Hydroalcoholic solution, delicately perfumed, intended for quick non-washable textiles disinfection and any other surfaces.

Effective against enveloped viruses like Coronavirus and its variants, H1N1 etc.

Recommended within the medical field (hospitals, medical institutions or veterinary field), communities, offices, public places or domestic. Suitable for disinfection of textiles as seats, sofas, chairs, stretchers etc.

Use adapted for disinfection in the field of cars and motorcycles : inside the helmet, shoes and interiors of vehicles (automotive seat, dashboard etc.). Use on objects at risk of manual transmission : door handles, phones, switches, computer equipment (mouse, keyboard), etc.

1 x 300mL aerosol can treat up to 10m².

INSTRUCTIONS FOR USE

Hold 20 cm away from surface and spray evenly. Leave to dry without wiping to allow the product to work.

PRECAUTIONS OF USE

For delicates surfaces, spray beforehand on a microfibre to a proper disinfection.

PACKAGING

Packaging unit: Aerosol can

Sales unit: Box of 12 units

Aluminum can Diameter: 53 mm/Height: 205 mm

Nominal volume: 405 ml/Net volume: 300 ml

Pallet size: 1512 units – 126 cartons

REGULATIONS

FOR INDIVIDUAL AND PROFESSIONAL USE

Biocide category: TP2 surface disinfectant, Dangerous. Follow precautions for use.

Use biocides with caution. Read the label and other information about the product before use. Refer to the safety data sheet (SDS). Available on request.

Our data sheets are prepared with current knowledge and are subject to modifications. Photo non-binding.

09/06/2022 Indice 3.5