



Ref.: A22469 HAND DISINFECTION SPRAY 50ML KING DERM'ALPES

Hygienic and surgical handrub sanitizer.

- ✓ Clear and unscented
- ✓ No need to rinse or dry hands



DISINFECTANT PROPERTIES IN COMPLIANCE WITH STANDARDS AT 20°C, CLEAN CONDITION:

Standards	Strains	Conditions
Bactericidal in compliance with EN 13727	<i>Staphylococcus aureus, Pseudomonas aeruginosa, Escherichia coli, Enterococcus hirae</i>	In 30 seconds
Bactericidal in compliance with EN 13697		In 5 minutes
Hygienic handrub in compliance with EN 1500		3 mL in 30 seconds
Surgical handrub in compliance with EN 12791		2 x 3 mL in 2 x 30 seconds
Fungicidal in compliance with EN 13624	<i>Candida Albicans and Aspergillus brasiliensis</i>	In 30 seconds
Fungicidal in compliance with EN 13697		In 5 minutes
Limited spectrum virucidal in compliance with EN 14476:2015	<i>Murine norovirus, adenovirus and vaccinia virus</i>	In 30 seconds
Additional viruses in compliance with EN 14476:2015	H1N1	In 30 seconds
Additional viruses in compliance with EN 14476 + A2	Rotavirus	In 30 seconds
Mycobactericidal in compliance with EN 14348	<i>Mycobacterium avium complex and mycobacterium terrae (= tuberculocidal activity)</i>	In 5 minutes

APPLICATION

Hydroalcoholic solution, for external use, intended for quick hand disinfection. Use on clean dry hands. No need to rinse or dry hands. Recommended within the medical field (hospitals, medical institutions, dental care centres, school, kindergarten and nursery sick rooms) or veterinary field (surgery and veterinary clinics). Effective against enveloped viruses like coronavirus, H1N1 etc.

Effective against enveloped viruses like coronavirus, H1N1 etc.

INSTRUCTIONS FOR USE

Hygienic handrub: 3 mL on the hands and rub until it has completely evaporated (minimum 30 seconds).

Surgical handrub: 2 x 3 mL in 2 x 30 seconds. This means 3 mL (refer to page 2 of packaging) into hand and rub until it has completely evaporated (minimum 30 seconds). Then repeat.

Stage 1: Palm to palm.


Stage 2: Right palm on the back of left hand and left palm on the back of right hand.

Stage 3: Palm against palm with fingers interlaced.

Stage 4: Back of fingers against opposite palm with fingers interlocked.

Stage 5: Rotational rubbing of thumb clasped inside left palm and vice versa.

Stage 6: Rotational rubbing in a backward and forward movement with clasped fingers of right hand in left palm and vice versa. At the end of stage 6, repeat the sequence of the same stages appropriately until the specified washing time has ended.

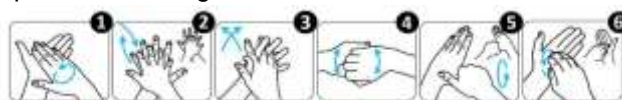

PACKAGING

Presentation: 50 mL aerosol

Aluminum can Diameter: 35 mm / height: 125 mm

Nominal volume: 110 mL / Net volume: 50 mL

Pack size: Box of 24 units

Pallet size: 4608 units – 192 boxes

REGULATIONS

For individual and professional use. Biocide category: TP1, human hygiene. Dangerous. Follow precautions for use. Use biocides with precaution. Read the label and other information about the product before use. Refer to the safety data sheet (SDS). Available on request. Our data sheets are prepared with current knowledge and subject to modifications. Photos non-binding. 16/01/2023 Index 3.3



Retrouvez toutes nos fiches techniques sur www.sico.net

SIÈGE SOCIAL / HEADQUARTER
577, rue du Pommarin B.P. n° 16 - VOREPPE
38341 Moirans CEDEX France

TEL : +33 (0)4 76 50 85 50 / commercial@sico.net

